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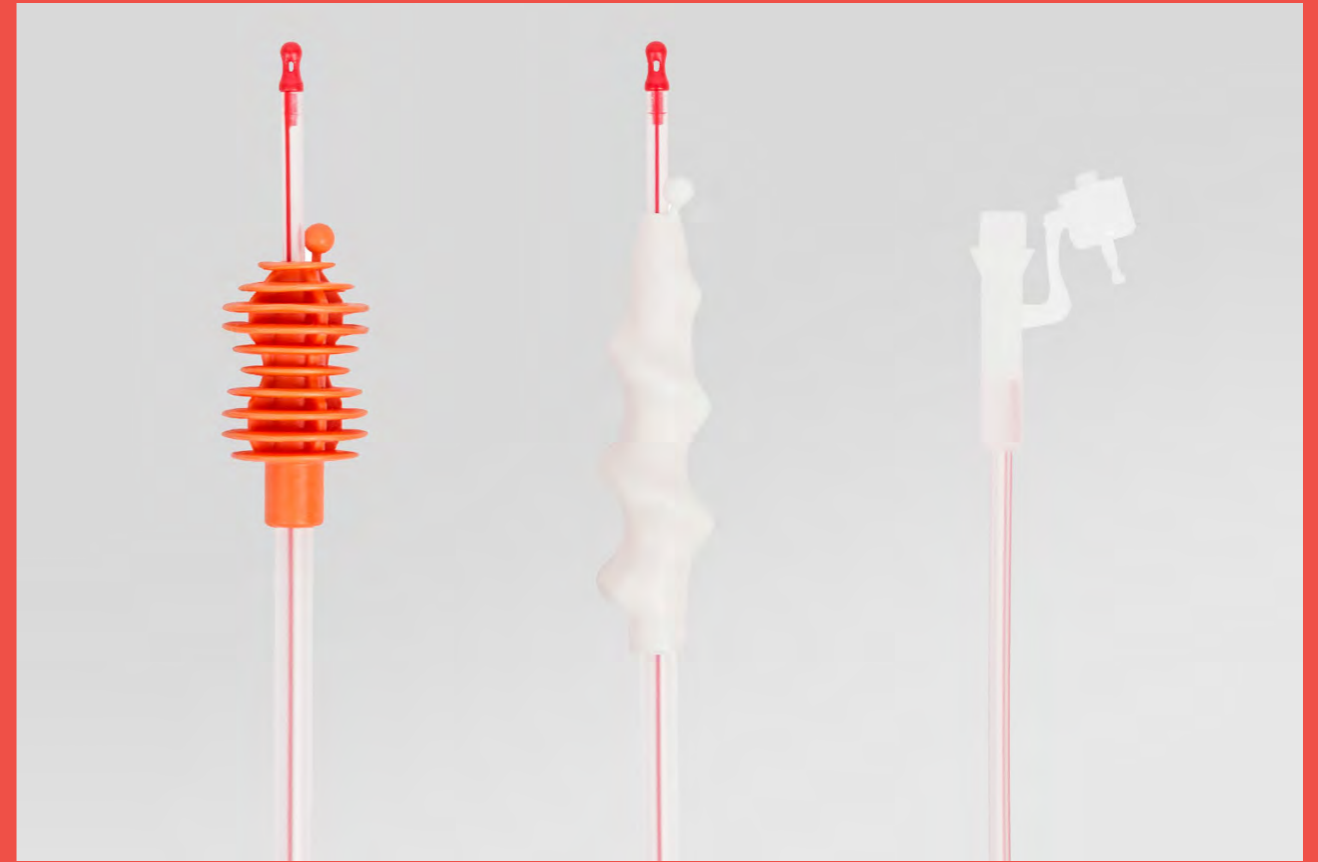
Once oestrus has been diagnosed (positive standing reflex) and the necessary number of hours have passed according to coverage protocol, the post-cervical insemination technique (PCAI) will be applied according to the following protocol:

1. Carefully clean the sow's vulva. We recommend the use of Tecnovet wipes.

2. Remove the catheter - cannula kit (SOFT & QUICK®) from its packaging.

3. Apply Tecnovet bactericide lubricant gel to the tip of the catheter.

4. Insert the catheter - cannula kit until the tip of the catheter fits into the cervix.

5. Hold onto the catheter with one hand and insert the cannula 1 to 2 cm with the other until the catheter safety stopper opens. The guide catheter includes a stopper that covers its exit hole to prevent the cannula from being contaminated as it is inserted into the vagina.

6. Turn the cannula until the red line is facing you. The cannula has a red line along its entire length to indicate the position of the exit holes on the cannula head.

7. Wait for one to two minutes before inserting the cannula. The procedure will be as follows in the work routine to avoid losing time:  
- When one person is inseminating: Prepare four sows, repeating steps 1 to 6 before returning to the first to continue with step 8.  
- When two people are inseminating: One person prepares five sows, repeating steps 1 to 6, and the other continues with step 8 on the first sow when the first person is preparing the fifth.

8. Pass the cannula gently yet firmly through the dilated cervix until it reaches the body of the uterus.

9. Once the cannula has been inserted, apply the dose of semen to the rear of the cannula and inject by squeezing the dose at storage temperature (15 -17 °C). Given that the exit holes on the cannula are located to the side of the red line on the cannula, the semen flows in the direction of the uterine horns for easier absorption.

The volume and concentration of the dose of semen for PCAI vary between 45 and 60 ml, and 1,500 and 2,000 million spermatozoa.

These doses can be produced using heat-sealed tubes, blister packs or bottles, or the semen can be packed into large aluminium/plastic complex bags holding 1, 2 or 3 litres, which protect it from the light and from temperature changes, to dose the semen using an adjustable dosage gun. This system is known as MULTIDOSIS SOFT & QUICK® and is ideal for medium and large-sized farms.

10. Once the dose of semen has been applied, remove the cannula by around 25 cm and, with the catheter still fixed to the cervix, perform a cervical massage in circular movements for 5 seconds. Once this is complete, immediately remove the catheter-cannula kit.

11. If the PCAI process has been successful, no immediate reflex will be observed after insemination. Where a significant volume of reflex is observed, repeat the process.

Post-Cervical insemination offers significant advantages:

1. Reduction in dose volume. 45 - 60 ml compared with 80 - 100 ml used in the cervical technique.

2. Reduction in the total number of spermatozoa per dose. 1.5 - 2 billion compared with the 2.5 - 3 billion used in the conventional technique.

3. Higher number of doses per ejaculate.

4. Reduction in time required for P.C.A.I.

5. Reduction in the volume required to carry and store doses of semen.

6. Reduction in the time required for insemination.

7. Improved operator working environment.

8. Reduction in the number of boars and in the necessary facilities.

9. Greater use of genetically superior boars.

10. Greater genetic uniformity of piglets.

11. Reduction in production costs per Kg. of meat.

REF. 10.320  
**Soft&Quick con catéter.**  
 Soft&Quick with catheter.  
**REF. 10.320.02**  
**Soft&Quick con catéter 2 en 2.**  
 Soft&Quick with 2-in-2 catheter.  
**REF. 10.320.03**  
**Soft&Quick con catéter 4 en 4.**  
 Soft&Quick with 4-in-4 catheter.  
**REF. 10.320.04**  
**Soft&Quick con catéter 25 en 25.**  
 Soft&Quick with 25-in-25 catheter.

REF. 10.320.30  
**Soft&Quick con catéter ibérico.**  
 Soft&Quick with catheter for iberico sows.  
**REF. 10.320.31**  
**Soft&Quick con catéter ibérico 2 en 2.**  
 Soft&Quick with 2-in-2 catheter for iberico sows.  
**REF. 10.320.36**  
**Soft&Quick con catéter ibérico 25 en 25.**  
 Soft&Quick with 25-in-25 catheter for iberico sows.

REF. 10.320.10  
**Soft&Quick con catéter esponja.**  
 Soft&Quick with sponge catheter.  
**REF. 10.320.13**  
**Soft&Quick con catéter esponja 2 en 2.**  
 Soft&Quick with 2-in-2 sponge catheter.  
**REF. 10.320.27**  
**Soft&Quick con catéter esponja 25 en 25.**  
 Soft&Quick with 25-in-25 sponge catheter.

REF. 10.320.40  
**Soft&Quick con catéter multianillas.**  
 Soft&Quick with multiring catheter.  
**REF. 10.320.41**  
**Soft&Quick con catéter multianillas 2 en 2.**  
 Soft&Quick with 2-in-2 multiring catheter.  
**REF. 10.320.42**  
**Soft&Quick con catéter multianillas 4 en 4.**  
 Soft&Quick with 4-in-4 multiring catheter.  
**REF. 10.320.43**  
**Soft&Quick con catéter multianillas 25 en 25.**  
 Soft&Quick with 25-in-25 multiring catheter.  
**REF. 10.320.32**  
**Soft&Quick con catéter ibérico multianillas.**  
 Soft&Quick with multiring catheter for iberico sows.

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Soft&Quick with M-10  
catheter for gilts.

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Soft&Quick cannula black  
line with rubber, silicone  
rear.

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Soft&Quick cannula black  
line with rings, silicone  
rear.

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Soft&Quick cannula black  
line with multirings, silicone  
rear.

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Soft&Quick cannula black  
line for iberico sows, silicone  
rear.

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Soft&Quick cannula black  
line with sponge, silicone  
rear.

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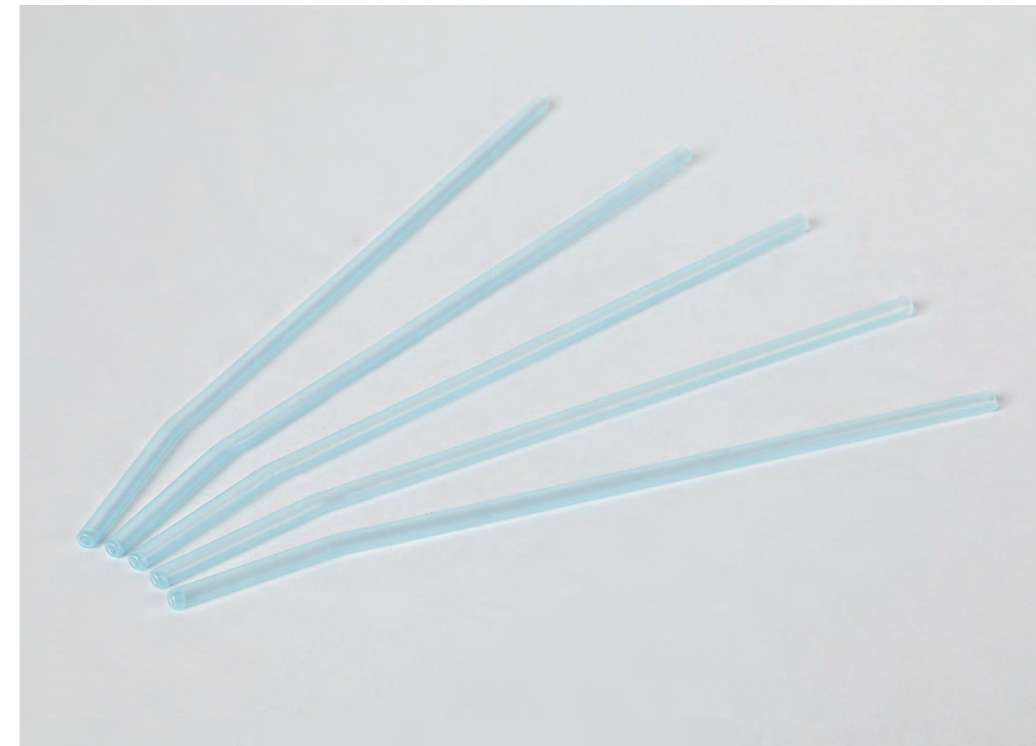
REF. 10.803  
Vagina artificial  
conejos blanda.  
Soft artificial  
vagina for rabbits.

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Sistema multidosis ©